WERO

Beatmungsmaske

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Pocket Mask

Operating Instructions

Masque de poche

Mode d'emploi

Maschera tascabile

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Máscara portátil

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DEUTSCH

Anwendung

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
ENGLISH

Application


The pocket mask was particularly developed for the purpose of mouth /mask respiration in case of breathlessness (for adults, children and infants). This device should be only applied by persons being instructed in the use of respiration assistance.

The pocket mask is provided with a direct connection for other respirators f.ex. resuscitator or oxygen-inhaling apparatus.

Operation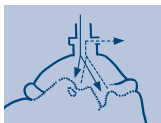

Expiration

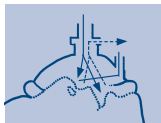
Oxygen

Operating principles for different applications are illustrated below.


Use for Adults and Children

1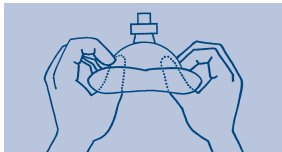

Take care of the exact position of the filter and press mask dome to the external side.

2

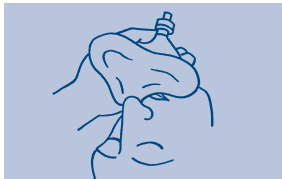Note

Instructions 2-3 to be applied only for adults and children! Place mask with the part for the chin between lower lip and chin of the patient. At the same time push back lower lip to keep mouth open. Put the small end of the mask over the nose of the patient.

3


Seal mask according to illustration. Open respiration channels. Blow carefully into the mask, until thorax lifts.

Note

In case patient vomits, take mask off and clear respiratory channels. Clean mask by watering or shaking. Blow through valve to ensure function. Replace mask and continue with respiration.


The mask can also be used without valve.

Use for Infants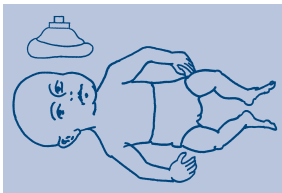

Follow the mentioned instructions with the exception that mask has to be placed on the face in the opposite side [narrow end of mask to be placed under the chin of the infant].

Application with Oxygen

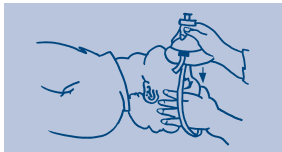Fix mask by means of head belt which will enable a better heart-lung resuscitation and direct inhalation.

Supplement oxygen being required can be added by using the extra oxygen connection placed at the mask. Stop flow of oxygen (see to chart). Respire, if necessary.

Characteristics

Temperature condition

Operation temperature: -18°C till 40°C

Storage temperature: -30°C till 50°C

Expiration and inspiration resistance

$< 3 \text{ cm H}_2\text{O} : 50 \text{ lpm}$

Deadspace $< 80 \text{ ml}$

Output data are nominal values.

Size of Mask connection

$\varnothing 26 \text{ mm}$

Size of Oxygen connection

$\varnothing 6 \text{ mm}$

Caution

The device is for single use only.
Reuse risks spreading infectious bacteria.
Sterilization and washing (for reuse) may
impair performance of the device and causes
harm for the patient.

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P.J. Dahlhausen & Co. GmbH,
WERO GmbH & Co. KG